sigvaris

Compreboot Plus

INELASTIC WRAP - FOOT

INSTRUCTIONS FOR USE AND CARE

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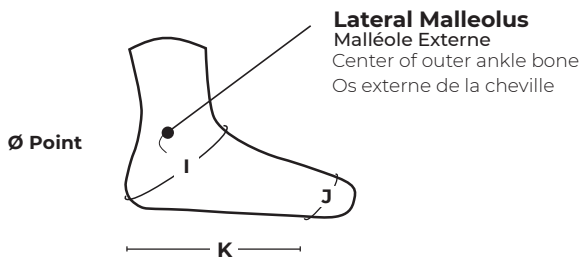
K: Measure length on the **medial** side of the foot from the heel to the 1st metatarsal head

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
Valid for Rest of the World

Product Measurement and Size Chart:

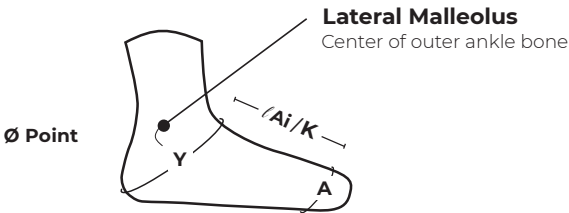Identifying the optimal product size ensures a proper fit and the correct distribution of compression. Measurements should be taken by a healthcare professional.

/Ai:
Measure length on the **medial** side of the foot from the heel to the 1st metatarsal head

***K (UK only)**

Compreboot Plus Foot Sizing Chart (A-Y)

	Small		Medium / Large		X Large / XX Large	
	Regular	Long	Regular	Long	Regular	Long
Y	28-36cm	28-34cm	39cm max	39cm max	44cm max	44cm max
A	22-26cm	22-25cm	30cm max	28cm max	33cm max	33cm max
/Ai/K	14-18cm	19-23cm	18-20cm	23-25cm	20-22cm	25-27cm

Compreboot Plus: Inelastic Wrap - Foot

MD

en	Medical Device
fr	<i>Dispositif médical</i>
es	<i>Producto sanitario</i>
it	<i>Dispositivo medico</i>
de	<i>Medizinprodukt</i>
pt	<i>Dispositivo médico</i>
pl	<i>Wyrób medyczny</i>
nl	<i>Medisch hulpmiddel</i>
da	<i>Medicinsk udstyr</i>
fi	<i>Lääkinnällinen laite</i>
ru	<i>Медицинское изделие</i>
lt	<i>Medicinos priemonė</i>
sl	<i>Medicinski pripomoček</i>
tr	<i>Tıbbi cihaz</i>
ja	医療用製品
ko	의료 기기
ar	الجهاز الطبي

Compreboot® Plus: Inelastic Wrap - Foot

Product Descriptor:

The Compreboot Plus Foot provides up to 40mmHg of foot compression and features soft, conforming neoprene fabric.

Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic edema. The product is intended for multiple use.

Indications:

Lymphedema, Edema, Chronic Venous Disease (CVD), Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis, Lipedema

Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia cerulea dolens
- Severe PAOD with any of the following: ABPI <0.6; ankle pressure <60mmHg; toe pressure <30mmHg
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- PAOD with ABPI >0.6 - <0.9 and/or ankle pressure between 60 and 90mmHg
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic edema
- Diabetes mellitus
- Skin diseases with massive exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or edema of the leg, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

Contents:

1 Compreboot Plus Foot

Material Composition:

- 44% Polyurethane Foam
- 30% Polyamide
- 22% Neoprene
- 3% Polyester
- 1% Elastane

Care:

- Close the velcro straps before laundering
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreboot Plus device should be washed as often as needed

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

Application:

1. Be sure the velcro tabs are open
2. With the garment tag at the back of the ankle and facing out, place your heel in the open heel space
3. Close the garment around the foot
4. Insert the straps into the D-ring and pull to create a snug, comfortable fit.
5. Readjust the straps as needed to ensure proper compression is maintained
6. The straps may be repositioned on the side of the garment to ensure a custom fit

Use:

While the Compreboot Plus Foot is suitable for day and nighttime wear, it should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the medical professional's recommendations. Sigvaris recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex**

Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.

Compreboot Plus: Inelastic Wrap - Foot

Product Descriptor:

The Compreboot Plus Foot provides up to 40mmHg of foot compression and features soft, conforming neoprene fabric.

Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic oedema. The product is intended for multiple use.

Indications:

Lymphoedema, Oedema, Chronic Venous Disease (CVD), Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis, Lipoedema

Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia coerulea dolens
- Severe PAOD with any of the following: ABPI <0.6; ankle pressure <60mmHg; toe pressure <30mmHg
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- PAOD with ABPI >0.6 - <0.9 and/or ankle pressure between 60 and 90mmHg
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic oedema
- Diabetes mellitus
- Skin diseases with massive exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or oedema of the leg, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

Contents:

1 Compreboot Plus Foot

Material Composition:

- 44% Polyurethane Foam
- 30% Polyamide
- 22% Neoprene
- 3% Polyester
- 1% Elastane

Care:

- Close the velcro straps before laundering
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreboot Plus device should be washed as often as needed

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

Application:

1. Be sure the velcro tabs are open
2. With the garment tag at the back of the ankle and facing out, place your heel in the open heel space
3. Close the garment around the foot
4. Insert the straps into the D-ring and pull to create a snug, comfortable fit.
5. Readjust the straps as needed to ensure proper compression is maintained.
6. The straps may be repositioned on the side of the garment to ensure a custom fit

Use:

While the Compreboot Plus Foot is suitable for day and night time wear, it should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the medical professional's recommendations. Sigvaris recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex**

Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.

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® Sigvaris and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014,
St. Gallen/Switzerland, in many countries worldwide. © 2021 Copyright SIGVARIS, Inc.
SIGVARIS AG
SIGVARIS GmbH